

Snappy™ Abutment impression technique

Quick guide

**This quick guide does not replace the instructions for use.
Please review the instructions for use before using the products.**


Step 1

- Place the Snappy Abutment onto the implant and hand-tighten the abutment screw with the Unigrip Screwdriver.
- Take a radiograph to verify proper seating.
- Tighten the abutment screw to 35 Ncm using the Unigrip screwdriver and the Manual Torque Wrench Prosthetic.




Step 2

- Align the arrow on top of the impression coping with the facial groove on the abutment. Push the impression coping onto the abutment. A “snap” will indicate that the impression coping is fully engaged and well adapted to the margin of the abutment.
- **Note:** If the Snappy Abutment has been modified, do not use the impression cap. Take a standard impression instead.

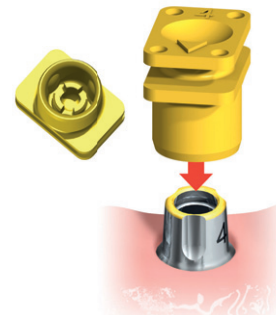

Step 3

- With the impression coping in place, inject an impression material (polyether material or polyvinylsiloxane) around the impression coping and into the tray.
- Record the impression. When the impression is removed, the impression coping will disengage from the Snappy Abutment and will remain in the impression.

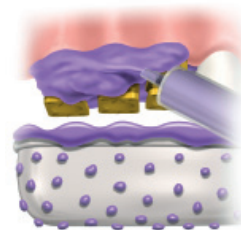

Step 4

- Place the healing cap or provisionalize the Snappy Abutment chairside using the plastic/temporary coping and a fabricated provisional restoration.
- Send the impression together with the Snappy Abutment Replica to the lab for crown fabrication.

