

Snappy™ Abutment impression technique

Quick guide

**This quick guide does not replace the instructions for use.
Please review the instructions for use before using the products.**


Step 1

- Place the Snappy Abutment onto the implant and hand-tighten the abutment screw with the Unigrip Screwdriver.
- Take a radiograph to verify proper seating.
- Tighten the abutment screw to 35 Ncm using the Unigrip screwdriver and the Manual Torque Wrench Prosthetic.




Step 2

- Align the arrow on top of the impression coping with the facial groove on the abutment. Push the impression coping onto the abutment. A “snap” will indicate that the impression coping is fully engaged and well adapted to the margin of the abutment.
- **Note:** If the Snappy Abutment has been modified, do not use the impression cap. Take a standard impression instead.

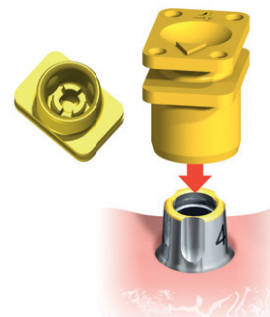

Step 3

- With the impression coping in place, inject an impression material (polyether material or polyvinylsiloxane) around the impression coping and into the tray.
- Record the impression. When the impression is removed, the impression coping will disengage from the Snappy Abutment and will remain in the impression.

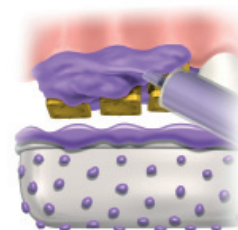

Step 4

- Place the healing cap or provisionalize the Snappy Abutment chairside using the plastic/temporary coping and a fabricated provisional restoration.
- Send the impression together with the Snappy Abutment Replica to the lab for crown fabrication.



GMT 40329 GB 1503 Printed in Sweden © Nobel Biocare Services AG, 2015. All rights reserved. Nobel Biocare, the Nobel Biocare logotype and all other trademarks are, if nothing else is stated or is evident from the context in a certain case, trademarks of Nobel Biocare. Product images are not necessarily to scale. Disclaimer: Some products may not be regulatory cleared/released for sale in all markets. Please contact the local Nobel Biocare sales office for current product assortment and availability.